


DIR[®]

Luxi

8838

INSTRUCTION MANUAL

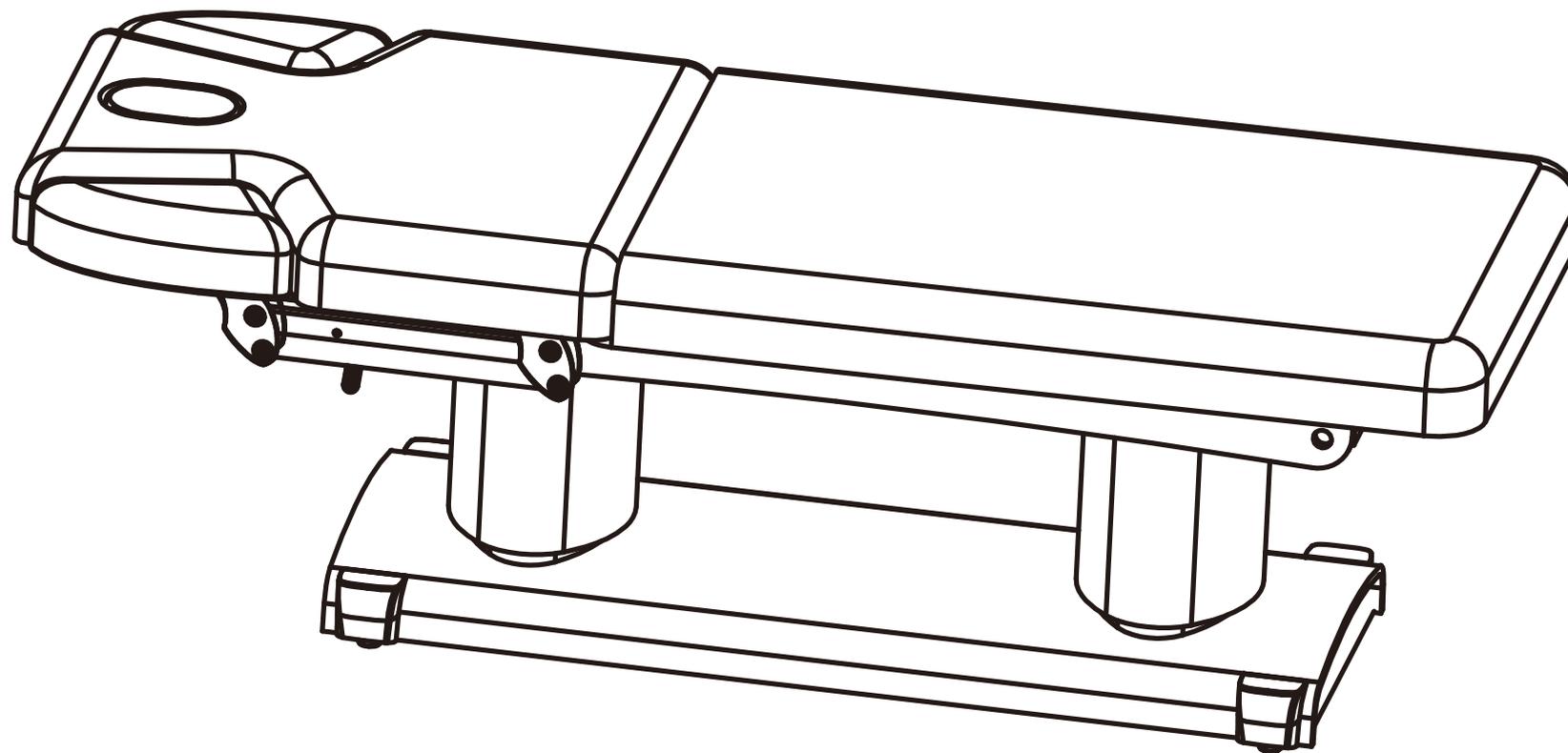



WARNING


- Power supply must be cut off when changing electrical components or maintaining and cleaning.
- The plug must be plugged into an outlet that is properly installed and grounded, to prevent the risk of fire and electric shock.
- Do not stand on this equipment.
- Do not allow children to play with and place the equipment in a place where children can touch. Please adjust the backrest to the highest position when you leave.
- The product should be kept in dry places and away from water / All electrical accessories such as remotes should be kept dry and free of water.
- Please do not place and operate the equipment in places where is painting/coating or supplying oxygen.
- Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lack of experience, common sense or children, are not allowed to use the equipment.
- Please contact distributor, service center or related personnel with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
- Operation should be processed by trained professionals.

PRECAUTIONS

- The equipment should be used by an experienced person who must be familiar with the product structure, performance and instructions. And make the regular maintenance work.
- Read this manual carefully before installation, operation and adjustment. And please keep it well.
- This product shall not be used for single side or single point loading work.
- Comply with safety operation rules, do not overload the use of products, to ensure that the product safety protection equipment is complete and reliable, eliminates unsafe factors on time.
- Tighten bolts and nuts regularly to avoid failure caused by loose, sliding, vibration and falling parts.
- During the maintenance, the operator shall not disassemble any parts of the bed at will.
- This product is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.
- The operator must be familiar with the precautions described in each section of the manual to ensure the safety of person and product.
- This electric beauty bed is equipped with a three-wire power cord. The power cord must be plugged into a three-hole outlet which is properly installed and grounded. This is not only to prevent the risk of electric shock, but also ensure bed work properly.
- Please operate the equipment strictly accordance with the warning label pasted to the products.
- While controlling the lifting of the whole bed, the button must be pressed continuously until the bed in a suitable position.
- Please pay attention that the continuous working times should be Max. 2Min. ON 18Min. OFF, when operating the motor.
- Maintenance should be processed by trained professionals.
- After each treatment, please use neutral reagent to clear the dust, stains on leather or on plastic cover.
- When adjusting the headrest, backrest, seat cushion, footrest or lifting of the whole bed by hand controller or installed foot controller, please keep your hands, feet and head away from the position.

	USA	UK
Net Weight	227 lbs	103 kg
Max Load Capacity	440 lbs	200 kg
Input Voltage	AC 110V/60HZ	AC 230V/50HZ
Output Voltage	DC 24V	DC24V

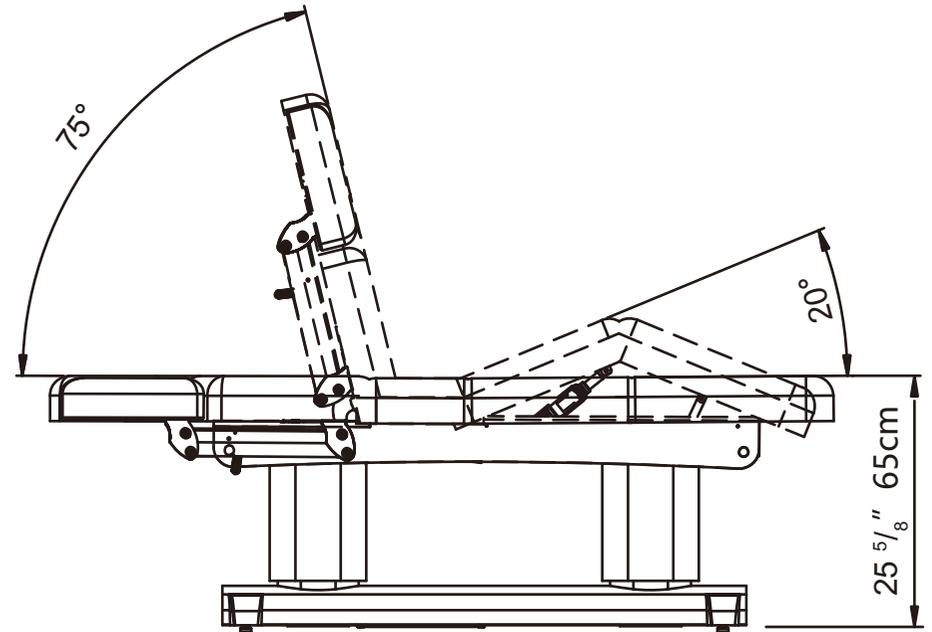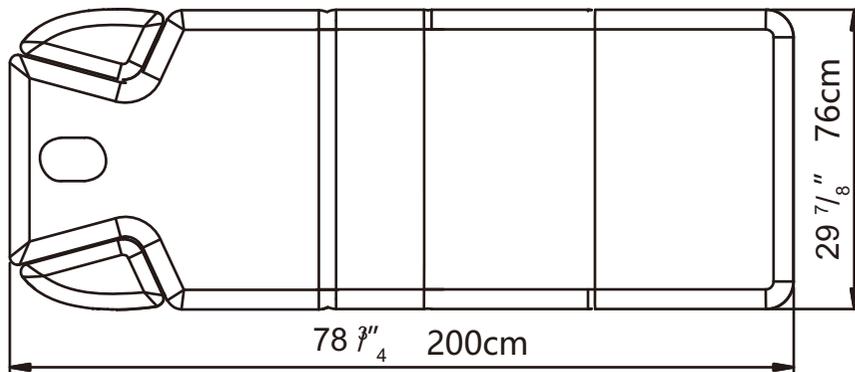

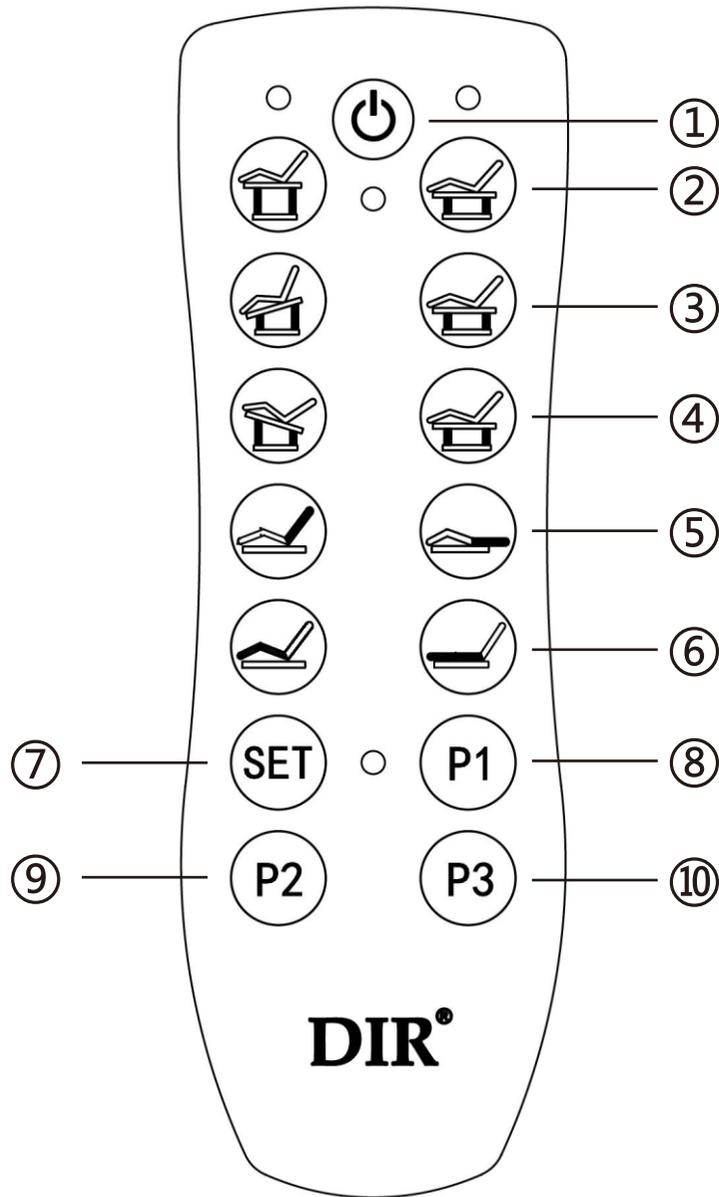

Press button ① to power ON the bed and be able to use all controls. Press ① again to power OFF the bed and the bed will return to its original state.

Press button ② to adjust the whole bed up and down. Press button ③ to adjust the incline and decline of the upper portion of the bed.

Press button ④ to adjust the incline and decline of lower portion of the bed.


Press button ⑤ to adjust the backrest up and down.

Press button ⑥ to adjust the footrest up and down.


Press button ⑦ press and hold SET button to activate memory configuration.

Press button ⑧-⑩ for your pre-set preferred height & chair position, up to three settings.



Height Adjustment

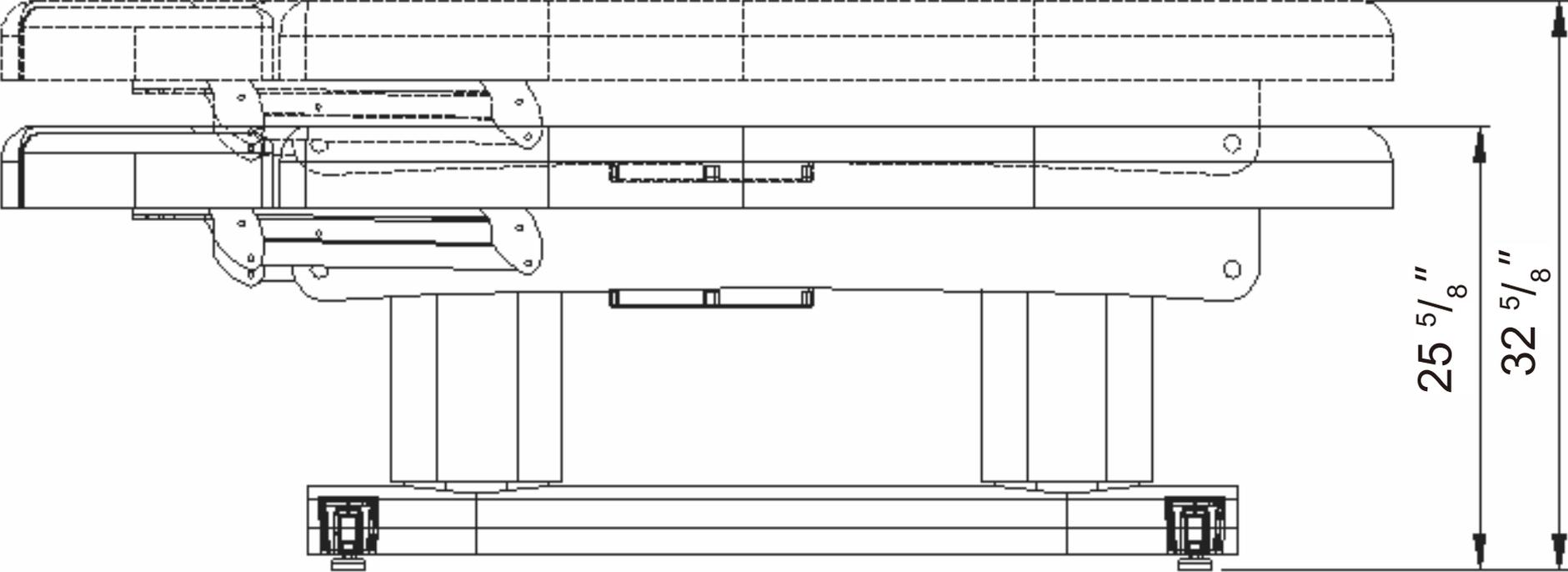

Backrest Adjustment


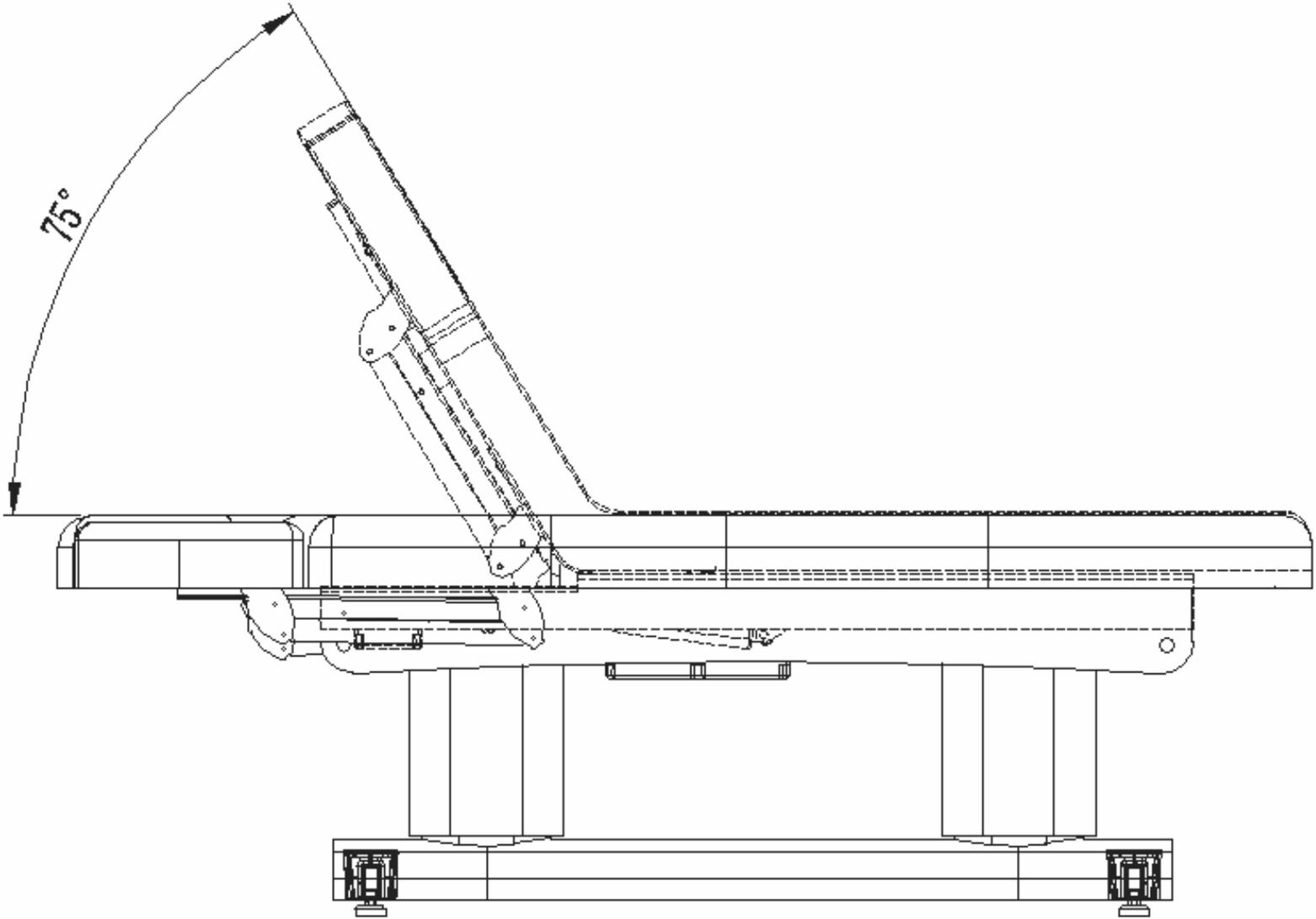


Footrest Adjustment

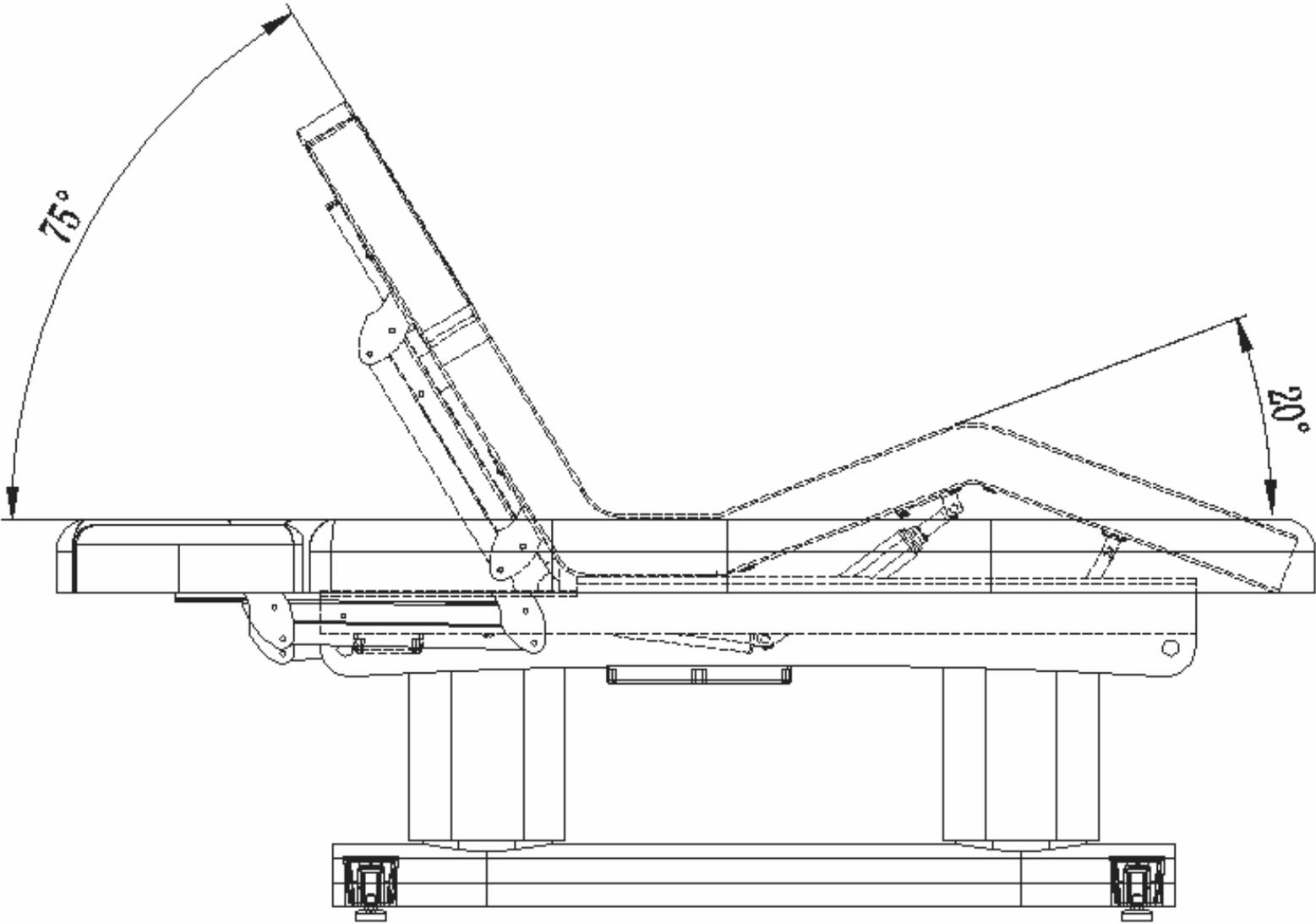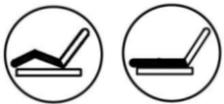

Upper and Lower Section Incline Adjustments

Trendelenburg & Reverse Trendelenburg Positions

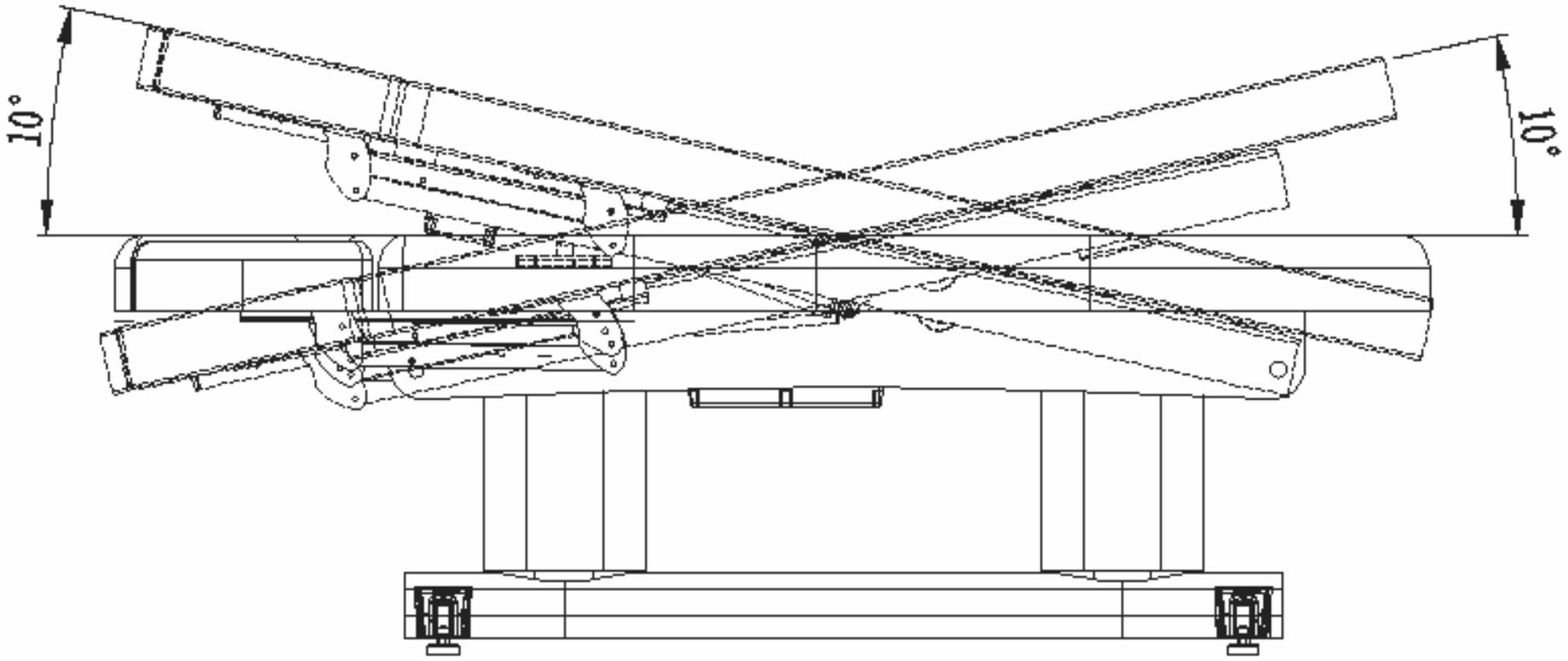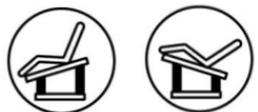


Maintenance & Care

1. Use damp towel to wipe and clean surface.
2. Simple soap and water can be used to wash surface.
3. Avoid using abrasive cleaning agents.


Product care




Brush away excess hair

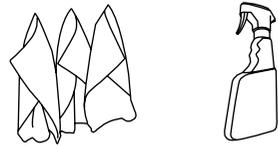

Clean surface with damp cloth or use simple soap & water solution.

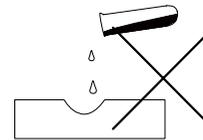

Avoid using corrosive cleaning agents such as bleaches.

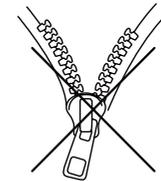

Do not remove upholstery for cleaning

Technical Troubleshooting Guide

Problem	Possible Cause	Solution
<p>*Unresponsive button controls</p> <p>*Beeping sounds</p>	<p>1.Thefunction of con trol button has reached to maximum.</p> <p>2.To check if the power plug is connected.</p> <p>3.To check if each connecting plug of the transformer is connected.</p>	<p>1.Try to press other control button.</p> <p>2.Disconnect the power plug and connect it again.</p> <p>3.Disconnect and reconnect all ports to the transformer.</p>

Warranty

The warranty period is one year. DIR warrants to you, the original buyer, that goods shall be free of defects in material and/or workmanship for one year from date of delivery.Damage resulting from improper handling after merchandise is received, or from exposure to weather or extreme atmospheric conditions is not covered by the warranty.

For any other information please contact our support team:

DIR BIRMINGHAM

66 Rea Street South
Birmingham
B5 6LB
UK

Tel:
+44(0)121 6434777

Email:
dirstalon@gmail.com

DIR LONDON

Unit 1 River Road
Business Park
33 River Road, Barking
Greater London
IG11 0DA
UK

Tel:
+44(0)20 8591 9310

Email:
dirstalon@gmail.com

DIR NEW YORK

575 Underhill Blvd.
Suite 170
Syosset, NY 11791
USA

Tel:
+1 (718)628-6668
+1 (800)728-2008

Email:
sales@dirstalonfurniture.com